BalanceTest is an approved in vitro diagnostics product for personal blood sample collection at home.

- First wash your hands with soap, rinse well with warm water and dry them.

Take out the sample card from the paper envelope. Save the envelope for later use.

Tear off the **SAVE** part on the sample card and take a picture of the Test ID. You can **ONLY** see YOUR test result with your anonymous Test ID. Place the card with the two circles facing up on the table.

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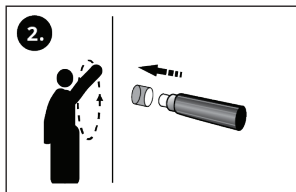
Use alcohol wipe to clean the tip of your finger (middle finger is recommended).

Stimulate blood flow by making big circles with your arm or shaking the hand downwards for 20 seconds.

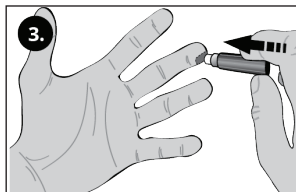
Take out the single-use lancet. Remove the transparent safety cap and the lancet is ready to use.

NOTE: Lancet is not included in the package. You may wish to purchase from recommended approved agencies in your local country.

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
Place the lancet against the lower part of the fingertip facing the collection paper on the table. Push the top of the lancet towards the finger until you hear a click. The lancet will automatically make a small prick in the finger.

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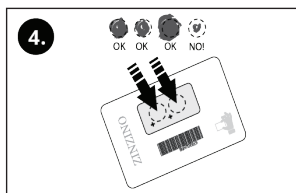
Don't touch the filter paper circles with your hands. Squeeze the finger gently one time and add several, minimum 3, free dripping blood drops to each circle marked on the sample card. The blood should completely fill the inner part of the circles.

Important: Avoid many aggressive squeezing of the finger to force the blood to drip.

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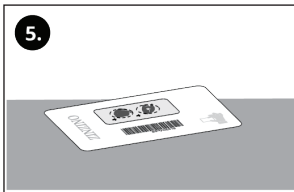
5.

Leave the **sample card** in a horizontal position in room temperature for at least 10 minutes for the samples to dry well.

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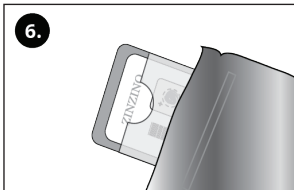6.


Insert the **sample card** back into the paper envelope. Then place the paper envelope into the metal bag and close it.

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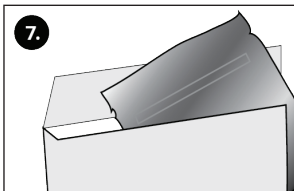7.

Place the closed metal bag into the big envelope with the laboratory address on it.

NOTE! You **MUST** put the correct amount of postage stamps on the envelope before you put in the mail box.

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8.

Go to internet and register your test code on www.zinzinotest.com. This is the web page where you can see your test results later.

It takes 10-20 days until your result is ready.

Important: Keep the SAVE part of the card. You can **ONLY** see YOUR test result on the internet with your anonymous Test ID.

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